

HemaGel[®] PROCTO

Suppository

INTENDED USE:

HemaGel[®] PROCTO is designed to support the healing of acute and chronic wounds of the rectal mucosa in patients with fissures and hemorrhoids, including patients after the Barron Ligature. Furthermore, to reduce bleeding in patients with bleeding hemorrhoids and to remove discomfort, especially burning and itching in the rectum.

COMPOSITION:

HemaGel[®] PROCTO is a white, torpedo-shaped suppository, containing powdered co-polymer Hema Has and adeps solidus fatty base.

TARGET GROUP AND LIMITATION OF USE:

Suppositories are intended for adult patients. Please consult a doctor before using it in children or in pregnant or breastfeeding women. With rectal disease, it is especially important to take care of hygiene and take care of regular stools. It is therefore necessary to drink enough fluids, eat a little spicy food and increase the intake of fiber.

HOW TO USE THE SUPPOSITORIES:

- Suppositories should be administered rectally, with the rounded end first, if possible after a bowel movement and after cleaning the anal area.
- Separate one suppository from the blister strip along the perforated edge, then tear the packaging by gently pulling it a part and remove the suppository.
- Insert the suppository deep into the rectum, rounded end first; this is best done in a slightly bent-forward position.
- Unless advised otherwise, **HemaGel[®] PROCTO** suppositories are administered once daily, best before going to bed.

CONTRAINDICATION:

Hypersensitivity to any of the components of the product, insufficient rectum length (in children under 3 months of age or preterm infants). Other contraindications are relative, so an application of a suppository should be consulted with the doctor in the following cases: constipation; diarrhea; serious illness, severe bleeding or post-traumatic condition in the rectum and rectum; anorectal disease such as abscess or fistula; abdominal resection.

IMPORTANT NOTES:

In the event of side effects, discontinue treatment and contact the attending physician. Protect yourself from children! Do not use after the expiry date stated on the container. Expiry date refers to the last day of that month. Dispose of your unused or expired remedy to your pharmacy. Do not take up to 30 days. The manufacturer recommends that if there is no difficulty within 7 days or if it gets worse, talk to your doctor. The suppository is for single use and rectal application. Do not apply to other body openings. Do not swallow. People with increased sensitivity may experience slight itching or burning in the rectum, which is not a malfunction but should be consulted with a physician.


STORAGE:

Keep at temperature 15-25°C, in the original inner packaging.

PACKAGE SIZE:

One package contains 5 or 10 suppositories, wrapped in molded PVGPE foil.

MANUFACTURED BY:

 VH Pharma a.s., Jakubská 647/2, 110 00 Prague 1, Czech Republic.

DISTRIBUTOR:

