

Hydrophilic gel.

HemaGel® NEW

By VH Pharma a.s.

INTENDED USE:

HemaGel® NEW is containing a polymer mesh intended for healing of both acute wounds, such as superficial injuries, including burns of I. and II. degree and postsurgical wounds, and also chronic wounds of a calf ulcer type, pressure ulcers and defects on the skin in patients with diabetes.

HOW HEMAGEL®NEW CONTRIBUTES TO THE TREATMENT OF WOUNDS?

- it chemically binds oxygen radicals, which forms during the process of wound healing
- it covers the wound against secondary infection from the outside
- it optimizes bacterial colonisation of the wound
- it forms an elastic mechanical protective layer on the skin
- it promotes the process of wound healing

COMPOSITION:

HemaGel® NEW is hydrophilic gel containing an IPN polymer protected by a patent, which has amines with steric shielding bound in a polymer form in its structure. It further contains macrogol 300 and purified water.

HemaGel® NEW is developed by the Institute of Macromolecular Chemistry of the Czech Academy of Sciences.

INSTRUCTIONS FOR USE AND APPLICATION:

- Clean the wound from mechanical impurities, e.g. dust particles, residues of previous medical materials, which could subsequently irritate and damage the wound.
- Distribute a thin layer of gel (approximately 1 mm) over the whole area of the wound base, slightly overlapping to a healthy skin. Gel will attach to the wound base and it gradually absorbs oozing and other fluids from the wound.
- Cover the wound with non-absorbent material (e.g. tulle gras available at the pharmacy) and subsequently fix it with a roller bandage or gauze square and adhesive plaster. Cover small injuries after application of gel with a cushion adhesive plaster.
- Rinse gently the residues of gel and impurities from the wound during change of dressing using a stream of water and then leave the wound to dry and apply a thin film of gel again.
- Change dressing as needed after 12 - 48 hours. Always according to the condition of the wound and oozing of secondary dressing or instructions of the attending person. The longest possible interval between changes of dressing is recommended in chronic wounds.

TARGET GROUP AND LIMITED USE:

The product is intended for all groups of patients with an acute or chronic wound.

CAUTION:

Discontinue the treatment in case of the incidence of adverse effects (increased pain, redness etc.) and contact your attending physician. Keep away from children! For external use only. Do not use internally! Do not use after the expiry date stated on the tube. Return the product to the pharmacy, if it is unused or after expiry date. Use in children, pregnant and breastfeeding women is recommended after consultation with the physician. Close the tube carefully after each use."

STORAGE:

Store in a dry place at 15-25 °C.

PACKAGE:

Tube 5 g, 30 g, 100 g.

MANUFACTURER:

■ VH Pharma a.s., Jakubská 647/2, 110 00 Prague 1, Czech Republic.

